

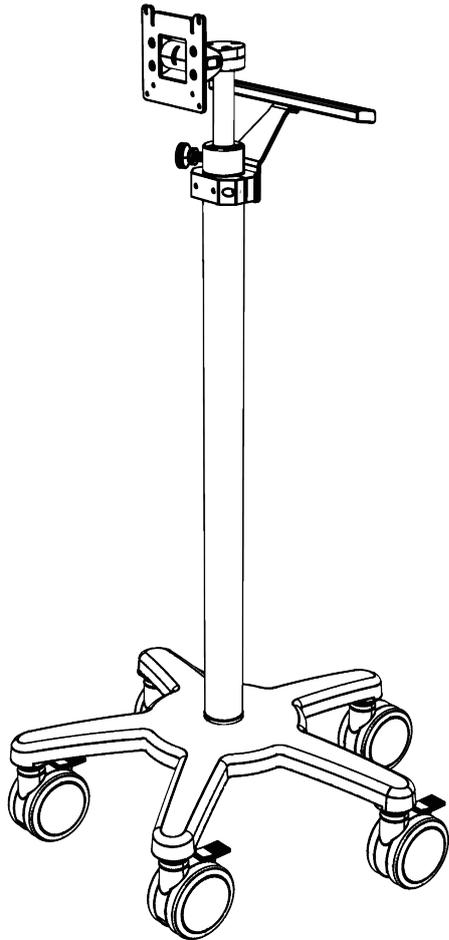
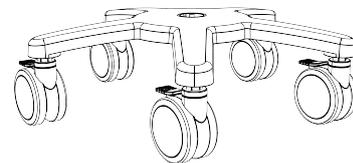
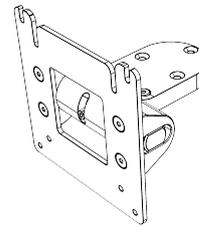
## MEDIplex 1 Cart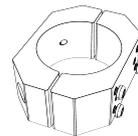

Interplex (Suzhou) Precision Engineering Ltd.  
 21 Lou Yang Street, CSS Industrial Park  
 Suzhou, Jiangsu Province, P. R. China 215122  
[rollstand@interplex.com](mailto:rollstand@interplex.com)  
[www.interplex.com](http://www.interplex.com)

**REF** 5000000037 / RS-007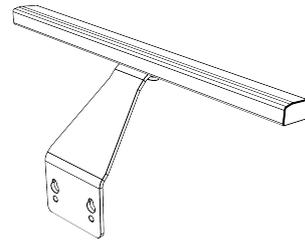

1  (1X)



2  (1X)

3  (1X)





4  (3X)


1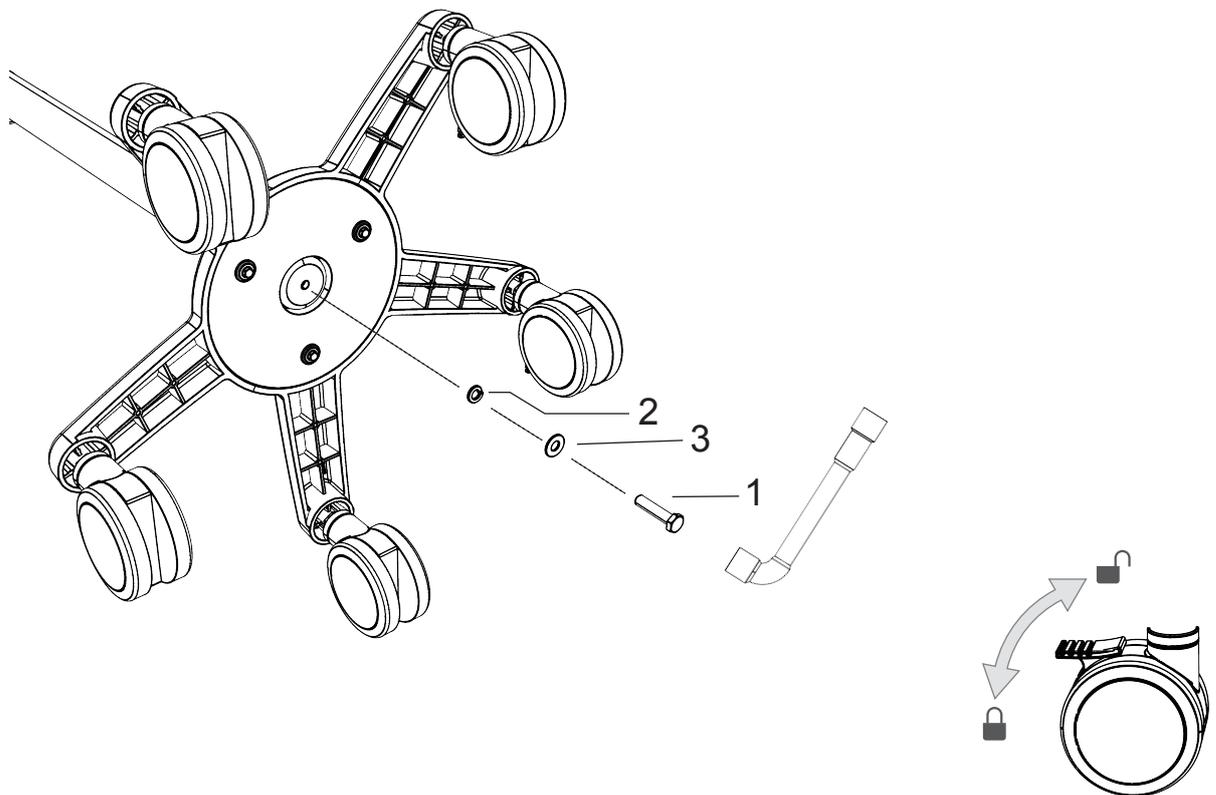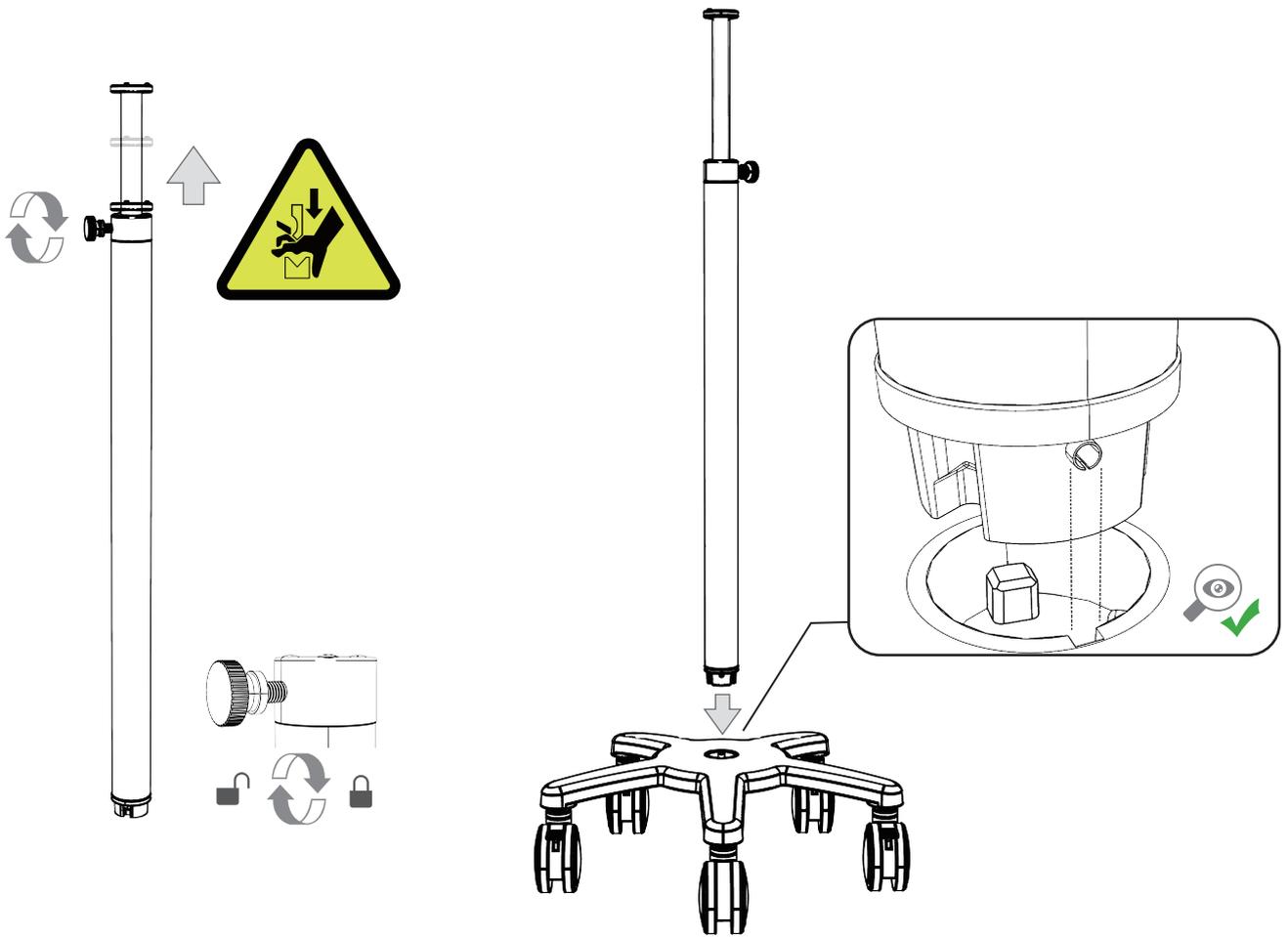



2

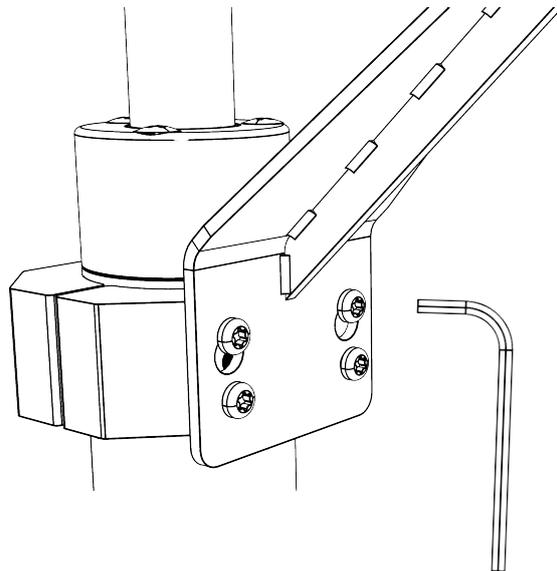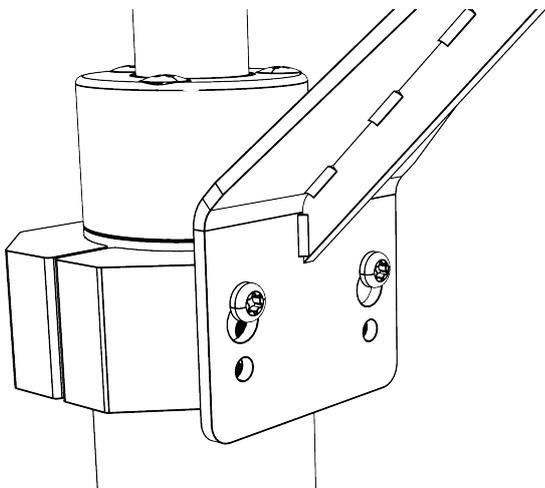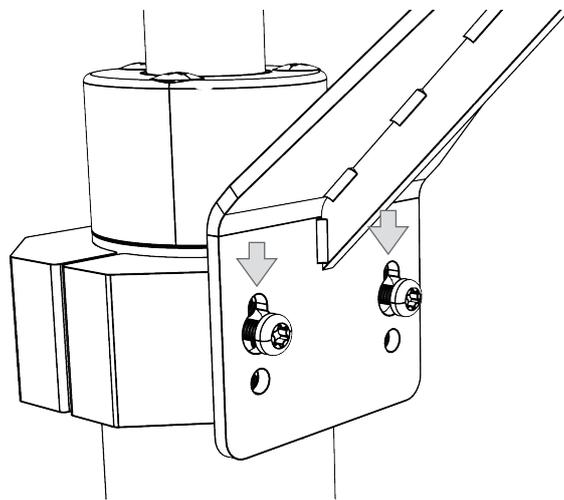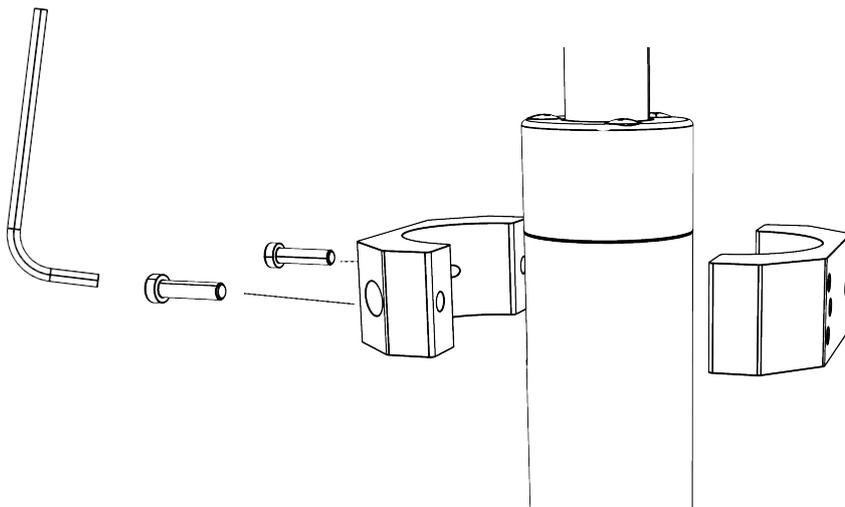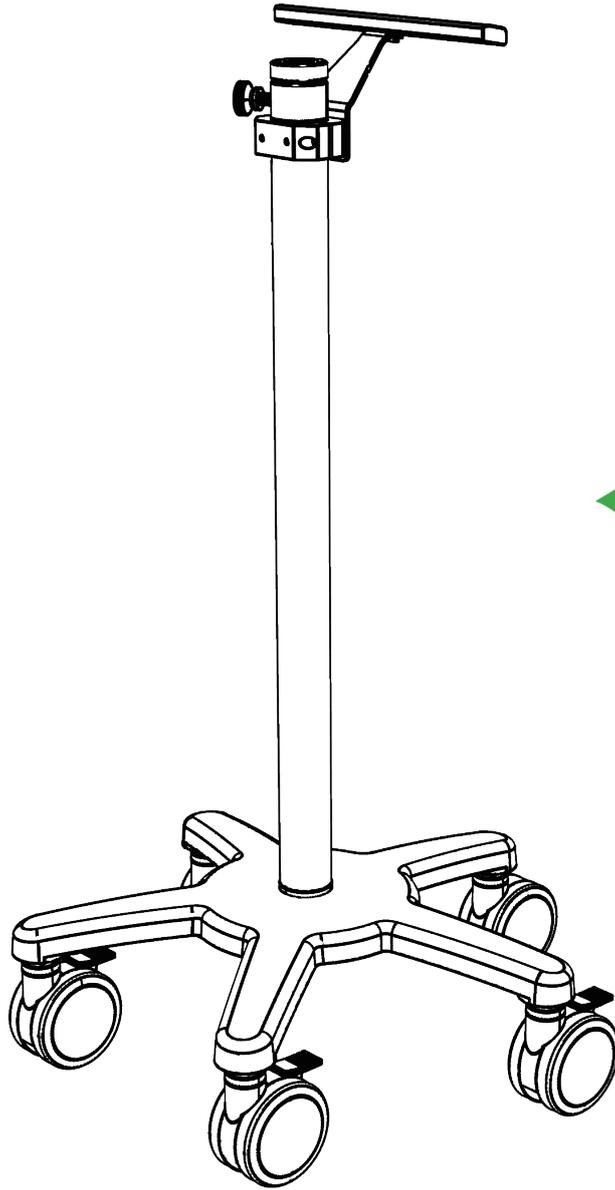
3

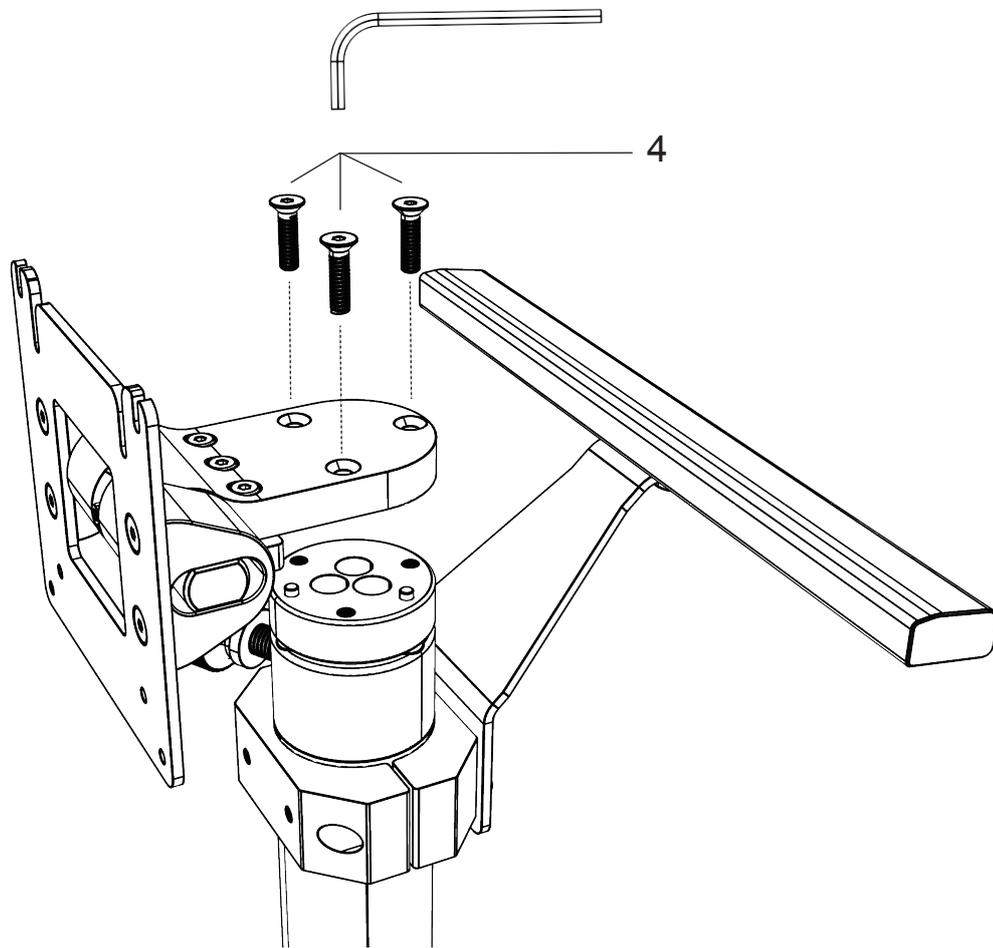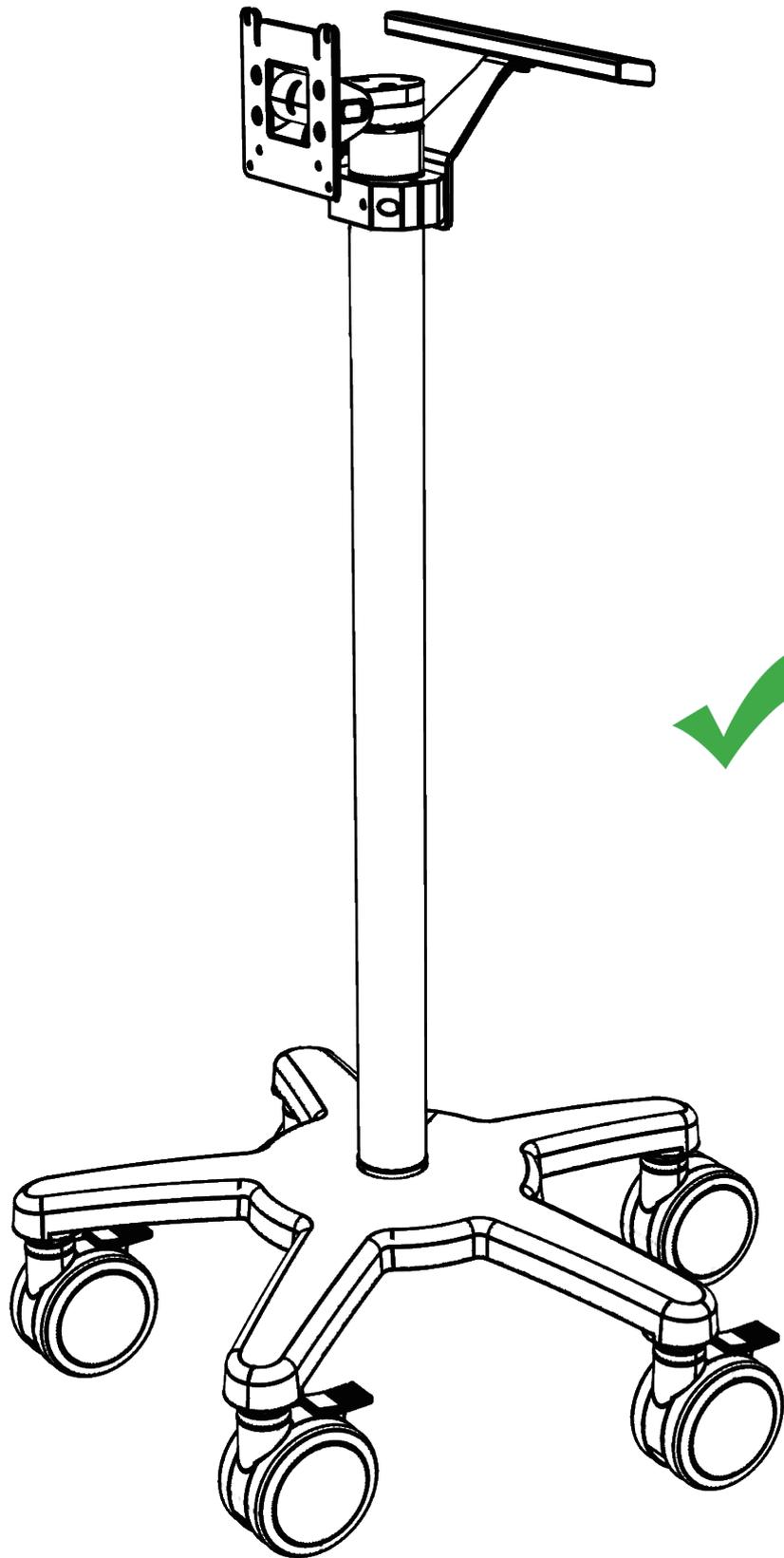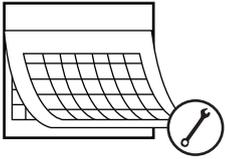
## Routine Maintenance

Periodically inspect all fasteners associated with the mounting system. Tighten or adjust fasteners as necessary for optimal operation and safety.

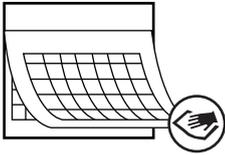## Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment.
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Do not use steel wool or other abrasive material.
4. Damage caused by the use of unapproved substances or processes will not be warranted. It is recommended that you test any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility. Refer to compatibility table above.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off the assembly immediately, using water dampened cloth. Dry the assembly thoroughly after cleaning.

**CAUTION:** Interplex makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment refer to the specific instructions delivered with those products.

## Chemical Compatibility


Mild detergent or soapy water  
 10% bleach solution (10%bleach,90%water)  
 70% Isopropyl Alcohol  
 70% Ethyl Alcohol  
 Hydrogen Peroxide(3%)

Acetone  
 Trichloroethylene  
 Steel wool or other abrasive material  
 Ethanol 100%  
 Oxivir® TB

## Fasteners and Tools Description

| NO. | Part Description                 |
|-----|----------------------------------|
| (1) | M8 x 25mm Hex head cap screw     |
| (2) | 5/16 Split Washer                |
| (3) | 5/16 Flat Washer                 |
| (4) | M5 x 20mm Countersunk Head Screw |

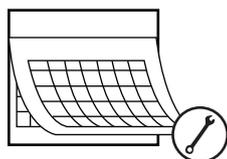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