

(English)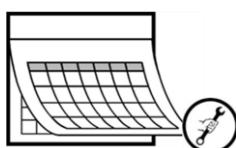

## Routine Maintenance

Periodically inspect all fasteners associated with the mounting system. Tighten or adjust fasteners as necessary for optimal operation and safety.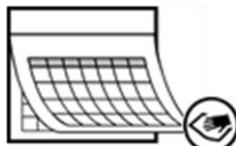

## Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment.
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Do not use steel wool or other abrasive material.
4. Damage caused by the use of unapproved substances or processes will not be warranted. It is recommended that you test any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility. Refer to compatibility table above.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off the assembly immediately, using water dampened cloth. Dry the assembly thoroughly after cleaning.

**CAUTION:** Interplex makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment refer to the specific instructions delivered with those products.


## Chemical Compatibility

Mild detergent or soapy water  
10% bleach solution (10% bleach, 90% water)  
70% Isopropyl Alcohol  
70% Ethyl Alcohol  
Hydrogen Peroxide (3%)

Acetone  
Trichloroethylene  
Steel wool or other abrasive material  
Ethanol 100%  
Oxivir® TB

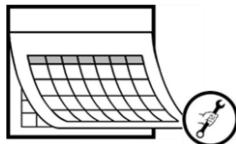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